

HOW TO USE BD™ ESwab™



Open & remove tube and swab applicator.



Collect specimen. Do not touch the area below the breakpoint line.



Aseptically remove cap from tube.



Insert swab into tube & bend shaft against tube at colored line to break shaft



Replace cap on the tube and secure tightly.



Apply patient identification to label.

COLLECTION AND TRANSPORT GUIDE

BD™ Liquid Amies Elution Swab (ESwab™)

For collection and transport of aerobes, anaerobes and fastidious bacteria.



220245 - BD™ ESwab™ Regular Collection Kit

White cap:

For nose, throat, vaginal and wound specimens.



220532 - BD™ ESwab Flexible Minitip Collection Kit

Blue cap:

For nasopharynx and pediatric specimens.

220246 - BD™ ESwab Minitip Collection Kit

Green cap:

For eye, ear, nasal passages, nasopharynx, throat, Urogenital tract and pediatric specimens.



BD Diagnostics
7 Loveton Circle
Sparks, MD 21152-0999
800.675.0908
www.bd.com/ds

The uses above are based on institutional procedures. See product package insert for manufacturer's recommendations.