

PROTEIN ELP, 24 HOUR URINE
218/LAB218

QUICK REFERENCE

On February 21, 2022, there will be a change to the specimen requirements for the Protein ELP, 24 hour urine (218/LAB218) assay.

DETAILS

	Current	NEW
Processing instructions	<ol style="list-style-type: none"> 1. Measure and record the total volume 2. Mix well 3. Transfer an 8mL aliquot to a Urine aliquot transport vial/tube – 10mL (AHL) 	<ol style="list-style-type: none"> 1. Measure and record the total volume 2. Mix well 3. Transfer a 20mL (<i>minimum 10mL</i>) aliquot into a <i>Screw cap plastic container (sterile)</i>
Volume	8 mL aliquot from a 24 hour urine collection	20 mL aliquot from a 24 hour urine collection
Minimum volume	5 mL aliquot from a 24 hour urine collection	10 mL aliquot from a 24 hour urine collection
Transport container	Urine aliquot transport vial/tube – 10mL (AHL)	Screw cap plastic container (sterile)

QUESTIONS: Contact your Allina Health Laboratory account representative, or our Client Services department