

Test update

Test name: Liver-Kidney microsomal (LKM) antibodies

Test #: 13414

Notification date: March 24, 2020

Effective date: April 06, 2020

QUICK REFERENCE

LabCorp has announced that effective April 06, 2020, the transport and stability requirements for the Liver-Kidney microsomal (LKM) antibodies assay (13414/LAB13414) assay will be changing.

DETAILS

	Current	NEW
Processing instructions	Spin, separate and freeze	Gold SST - Spin Red – spin and separate
Transport container	Screw-cap polypropylene frozen transport vial/tube - 4mL (LabCorp)	Gold SST or Transfer vial/tube with cap - 12mL (LabCorp)
Preferred temp	Frozen (<i>preferred</i>)	Ambient (<i>preferred</i>) – 14 days
Alternate temp	Refrigerated – 48 hours	Refrigerated – 14 days
Alternate temp	Ambient – 8 hours	Frozen – 14 days <i>Freeze/thaw cycle: stable x4</i>